



USER GUIDE

CLINICAL FEMALE PELVIC TRAINER (CFPT) MK 3

60900 STANDARD



FOR MORE SKILLS TRAINING PRODUCTS VISIT


limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SETTING UP THE CFPT	5
• CHANGING THE UTERINE MODULE	6
TRAINING	7
• BI-MANUAL EXAMINATION	7
CLEANING & STORAGE	7
• CLEANING & STORAGE AFTER USE	7

TAKE A LOOK
AT OUR **VIDEO**

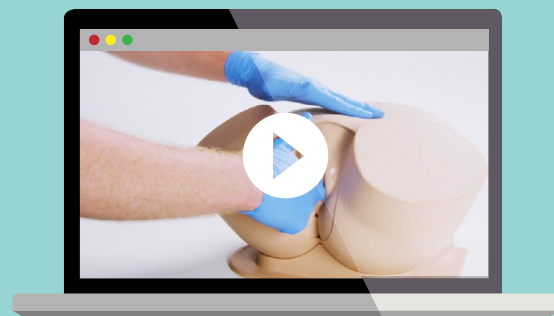

VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings


PACKAGE
SUPPLIED

60910


CFPT ABDOMINAL INSERT



LOWER TORSO

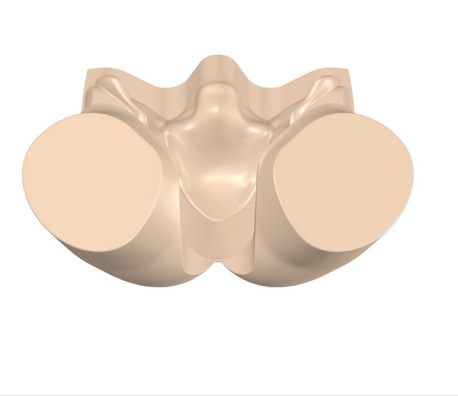

BASE

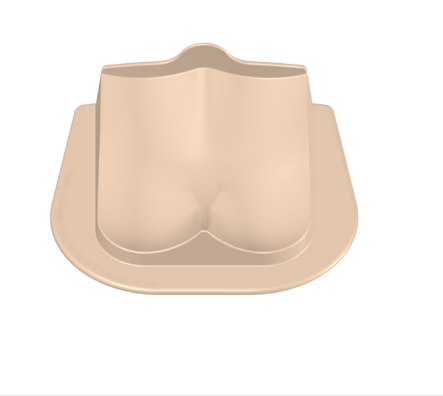

60920

CFPT MODULE 1

60920 CFPT MODULE 1

Normal - Nulliparous Cervix



The image contains two parts. On the left is a small, 3D anatomical model of a cervix, showing its external os and internal canal. On the right is a larger, 3D anatomical model of the female pelvis, showing the uterus, fallopian tubes, and ovaries in a sagittal view. The cervix is highlighted in a light pink color, and the uterus is shown in a light blue color. The diagram illustrates the normal anatomy of the cervix in a nulliparous woman.




00293

WATER-BASED LUBRICANT



CARRY CASE

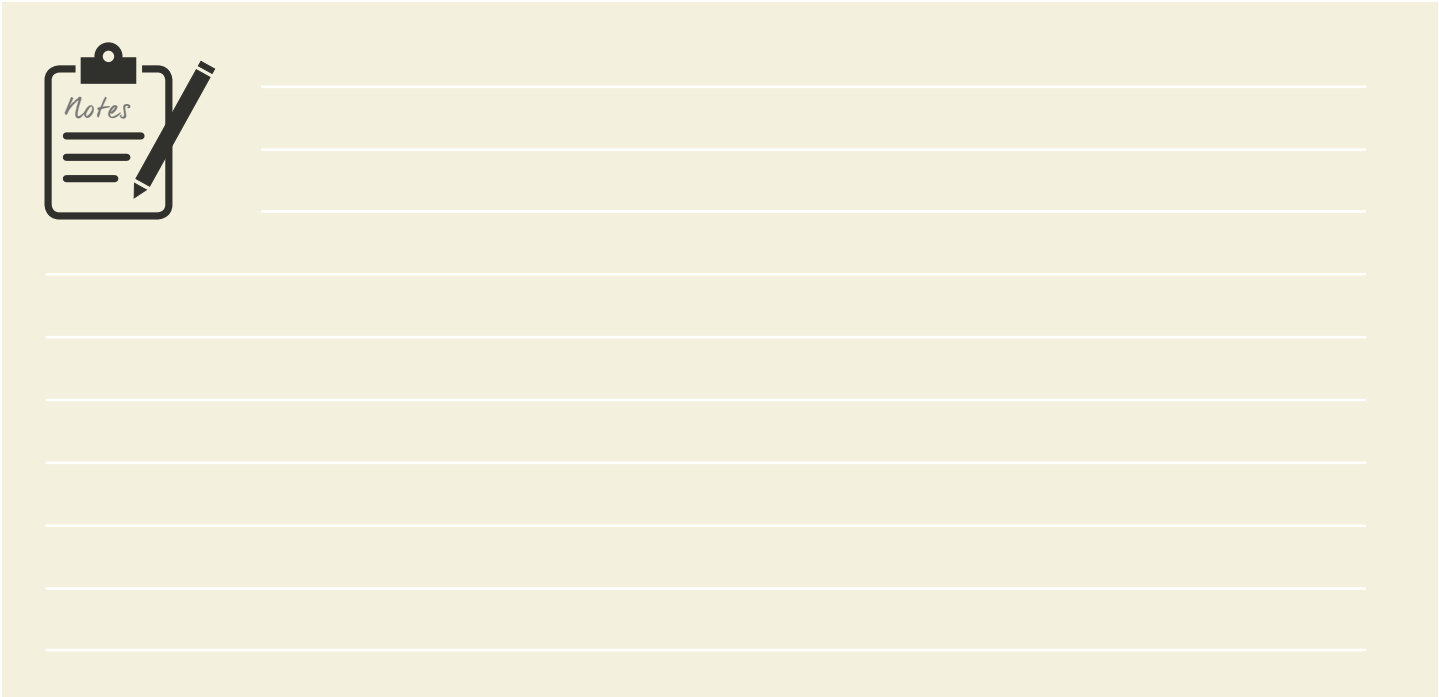

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR CLINICAL FEMALE PELVIC TRAINER

DO's



Follow all instructions.
Keep these instructions.
Heed all warnings.

Only use water-based lubricant, such as the one supplied with this model.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Lubricate your examining fingers before insertion.



Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.

STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

Do not use if the skin is torn.




Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.


SET UP

SETTING UP THE CERVICAL FEMALE PELVIC TRAINER

1

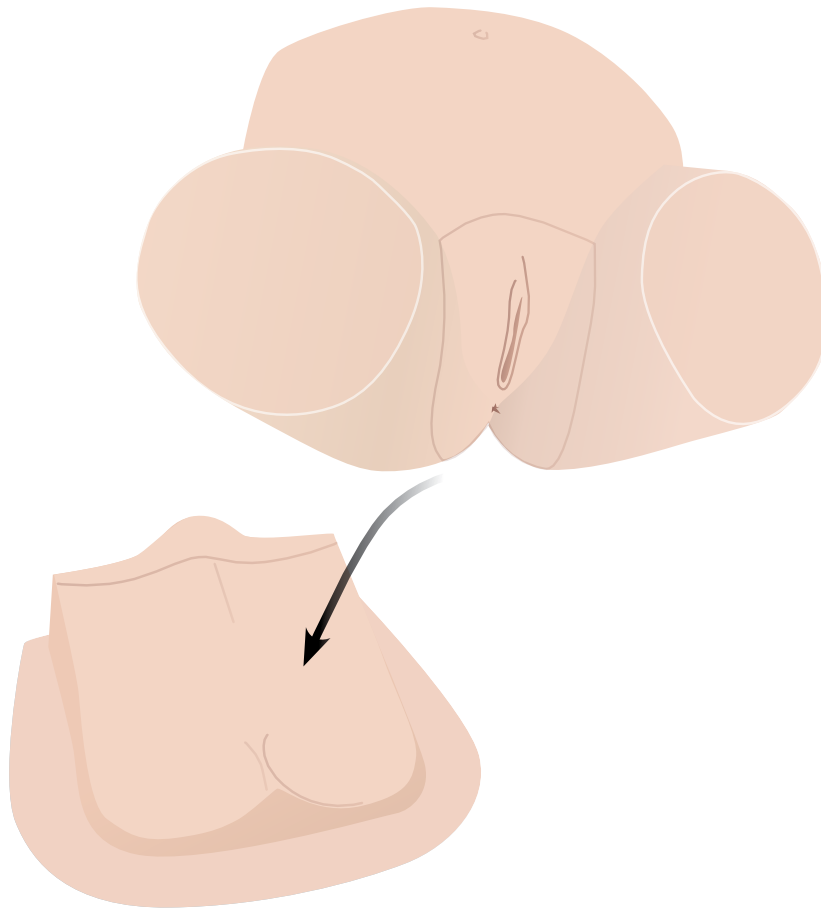

2

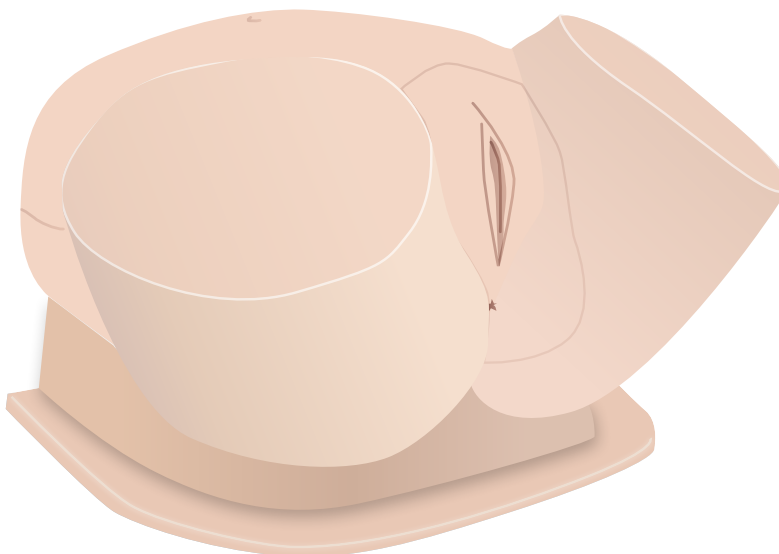


SET UP

CHANGING THE UTERINE MODULE

1




LIFT THE ABDOMINAL
INSERT UP FROM THE
TOP OF THE TRAINER

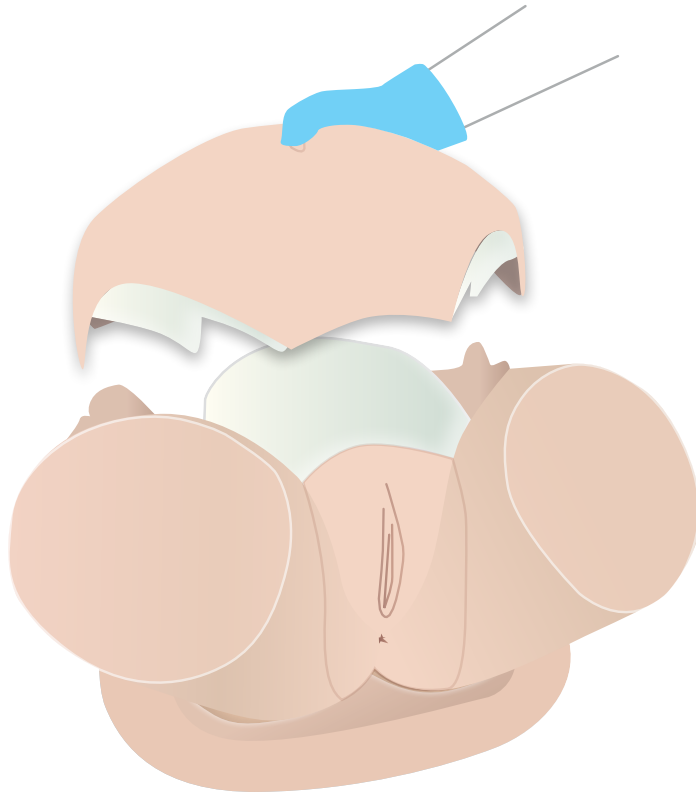

2




LIFT THE MODULE OUT
USING BOTH HANDS

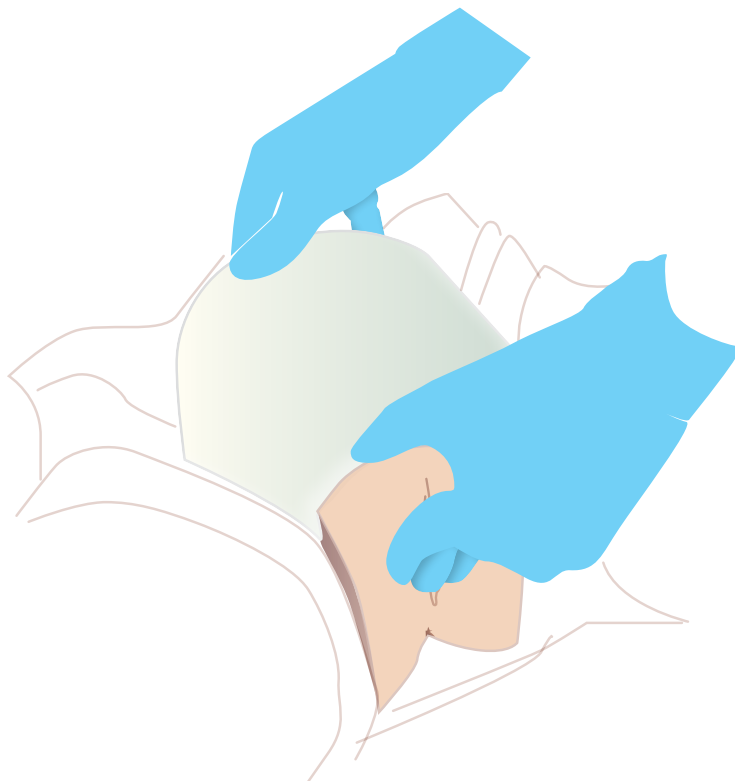

TRAINING

PERFORMING THE BI-MANUAL EXAMINATION


- Wear gloves at all times and remove all jewellery items.
- Lubricate prior to examination.

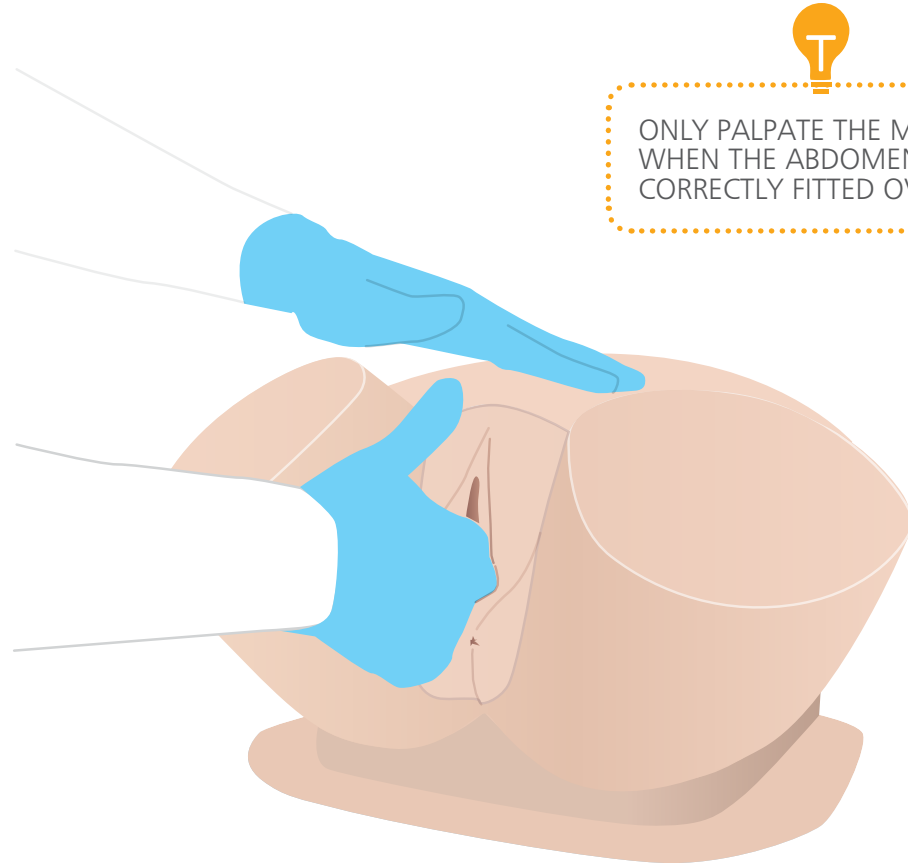

ONLY PALPATE THE MODULE
WHEN THE ABDOMEN IS
CORRECTLY FITTED OVER IT

CLEANING & STORAGE

- Wipe off excess lubricant with a soft cloth
- Clean off any surface dirt with a damp, soft cloth, using only warm water and mild detergent

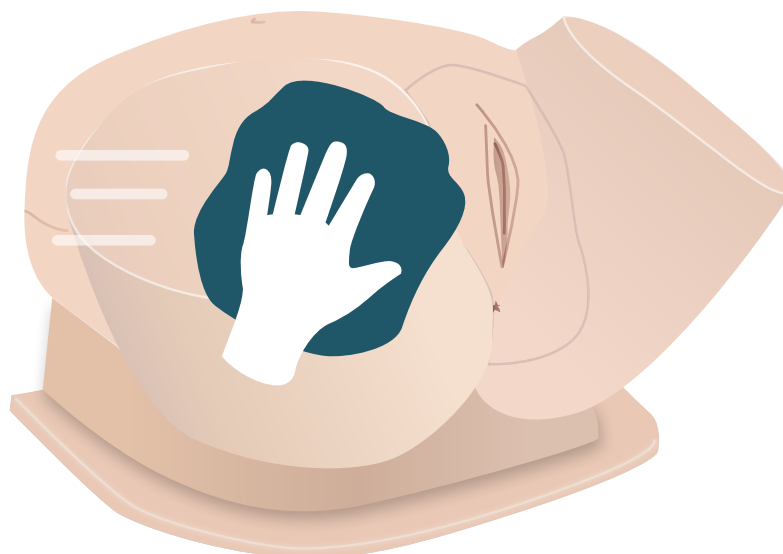

YOU MAY
ALSO BE
INTERESTED IN...

ADDITIONAL MODULES

60921 CFPT MODULE 2

Large Fibroid -
Nulliparous Ectropion Cervix




60922 CFPT MODULE 3

Small Fibroid -
Nulliparous Polyp Cervix




60923 CFPT MODULE 4

Ovarian Cyst - Multiparous Cervix

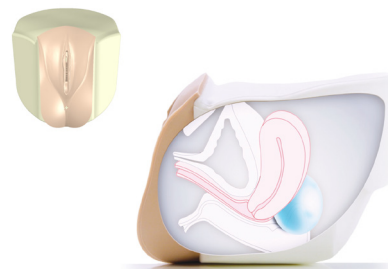


60924 CFPT MODULE 5

Retroverted - Multiparous Cervix

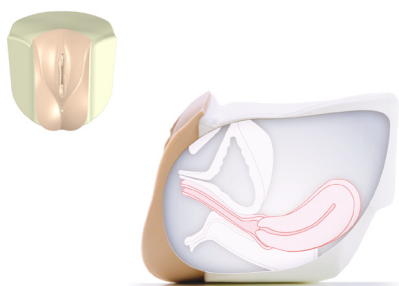


60925 CFPT MODULE 6

10-12 Weeks Pregnant

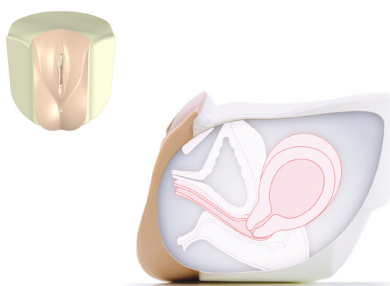

60926 CFPT MODULE 7

14-16 Weeks Pregnant

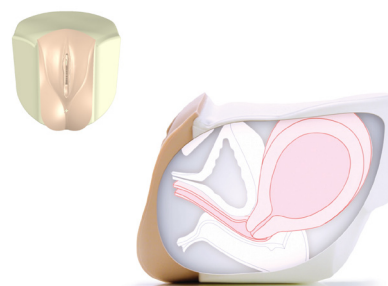

► Images above are for information purposes & show a simplified representation of anatomy within the CFPT Modules