## Directions for use Single patient surface electrodes

English/American

These directions for use are intended for the following Ambu Surface electrodes for neurophysiological examinations: Ambu Neuroline 700, 710, 714 (Ground), 715, 720, 721, 725, 726 (Cup).

### Product specific directions for use

**700 – 710 – 714 (Ground) – 715:** Prepare the skin using an abrasive. Degrease using alcohol wipe. Peel the electrode from its protective backing and apply. The electrode can be repositioned for other measurements.  
**720 – 721:** Prepare the skin using an abrasive. Degrease using alcohol wipe. Peel the electrode from its protective backing and apply. Press along the edges to ensure maximum adhesion. DO NOT press over the sensor area.  
**725:** Prepare the skin using an abrasive. Degrease using alcohol wipe. Apply the gel before use. Peel the electrode from its protective backing and apply.  
**726 (Cup):** Prepare the scalp using an abrasive. Apply gel, paste or adhesive on the cup electrode. Attach the cup to the scalp.

### General directions for use

- The correct placement of electrodes is typically determined by the hospital/physician and/or local protocol/procedures.
- The pre-attached lead-wire may be connected directly to the monitoring device.
- After monitoring remove the electrode from the skin (refer to directions for use on pouch).
- Gel residue can be removed using paper towels or water.
- The electrode is disposable and should be discarded according to hospital or physician guidelines.

Symbol	Indication
REF	Catalogue/Catalog number
LOT	Traceable number
⌚	Use by date. Electrode must be used before expiry/expiration date indicated on package
⚠	Do not reuse
⚠	Caution, consult accompanying documents
🌡	Temperature limitation. Store in unopened pouches at temperatures between 10°C/50°F and 25°C/77°F
🌡	Temperature limitation. Store in unopened pouches at temperatures between 10°C/50°F and 40°C/104°F
📄	Remaining electrodes should be stored in the pouch, properly closed, by folding over the open edge. Never use electrodes if the gel has dried out. For shelf life of unopened pouches please refer to the date of expiry/expiration printed on the pouch
🚫	No latex. Stated when products are latex free
🚫	No PVC. Stated when products are PVC free
⚠	Not sterilized

### Cautions

- Surface electrodes should only be used by or in consultation with experts. Careless removal of the electrode may cause damage to the skin. Should the electrode be difficult to remove use water to dissolve the adhesive.
- US: Rx only.

### Precautions

- Single patient use only. For monitoring and recording biopotential on the skin.
- During surgical procedures electrodes should be placed as far as possible from the electro-surgical area to minimize unwanted RF current flow.
- Electrodes in an opened pouch may dry out, therefore do not open pouch until necessary.
- Lead-wires are attached to electrode type 700, 710, 714 (Ground), 715, 725 and 726 (Cup). 720 is available with a choice of pre-attached lead wires and off-set press-stud. 721 is available with an off-set press stud.

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Ambu® on tanskalaisen Ambu A/S:n rekisteröity tavaramerkki.
Ambu A/S on sertifioitu standardien ISO 9001 ja ISO 13485 mukaisesti.