IMPORTANT: READ CAREFULLY
WICHTIG: SORGFÄLTIG LESEN

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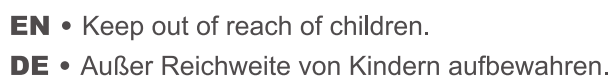
EN • BONYPLUS® 45 Powder 2.95g: Do not get in eyes, on skin, or on clothing. Rinse cautiously with water for several minutes. Dispose of contents/container in accordance with local regulation.

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6 700 01 A 0 0 19 - B08012018D-PT

EN

DENTURE RELINING CUSHION STABILIZER
Instructions for Use

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IF YOU ARE A FIRST TIME USER OF THIS PRODUCT, GO STRAIGHT TO POINT 1, PICTURE 1.1. IF YOU HAVE USED THIS PRODUCT BEFORE, PLEASE NOTE THE FOLLOWING: Remove the 30 day old Denture Relining Cushion Stabilizer by placing your denture in a glass of warm water (approx. 50°C) and soak for 10 minutes. Remove from water and peel away the product with the help of a blunt knife (if needed). Check that there is no prior glossy layer of the “BONYPLUS® 45 SEPARATOR” visible on your denture. If there is, do NOT apply another layer of “SEPARATOR” on this application. The function of the “SEPARATOR” is to enable the product to be removed easily at the end of the 30 days. If you have no use for the application of the Separator, you may discard it.

1 **Picture 1.1:** Clean and dry your denture.

Picture 1.2: Open the yellow coloured liquid labelled “BONYPLUS® 45 SEPARATOR” and stand it upright in the transparent plastic tray.

2 Using the small brush provided, apply thinly and evenly the SEPARATOR on the inside of your denture (the side of your denture that sits on your gums). To ensure a thin application check the glossy transparent appearance of the SEPARATOR.

Picture 2.1: Upper denture: keep 5 mm from the back and front inside edges (see shaded area on picture).

Picture 2.2: Lower denture: keep 5 mm from the front and inside edges.

Picture 2.3 IMPORTANT: If your lower gums are very flat and your denture has a small or no groove (see picture 2.3) do not apply the SEPARATOR at all.

Picture 2.4: Lower denture with normal or deep groove: apply SEPARATOR, also stay away 5 mm from the front and inside edges.

Allow the SEPARATOR to dry for 10 minutes before proceeding to next step.

3 **Picture 3:** Set the transparent plastic tray, brush and separator bottle aside. These can now be discarded.

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NOTE: When carrying out this next step, keep the blue mixing dish resting in the large white plastic holder to ensure a better hold and enable mixing to take place.

4 **Picture 4.1:** Remove the lid from the BONYPLUS® 45 powder 2.95g and pour the whole contents into the blue mixing dish. Remove the lid from the BONYPLUS® 45 liquid 2.5ml and add the liquid rapidly to the powder by gently tapping the bottom of the bottle with your finger.

Picture 4.2: Gently but thoroughly mix the powder and liquid using the blue spatula provided ensuring no lumps of powder are present in the mixture. This will take approximately 15-20 seconds. Immediately go to Step 5.

5 Pour the powder and liquid mixture immediately into the grooves of either the upper or lower denture. If any mixture remains in the blue mixing dish, use the blue spatula provided to remove it.

Picture 5.1: For upper dentures, pour the mixture into the middle of the denture to allow for easier spreading of the mixture using the blue spatula provided.

Picture 5.2: For lower dentures, pour the mixture inside the denture and spread evenly using the blue spatula provided. Immediately go to **Step 6**.

6 **Picture 6.1:** Place the denture in your mouth and tilt your head slightly forward to avoid the mixture running off the back of the denture into your throat.

Picture 6.2: Gently press your teeth together and hold for about 4 minutes. During this time breathe through your nose moving your cheek muscles and lips lightly with your mouth closed. After the 4 minutes has elapsed, gently remove the denture from your mouth.

7 **Picture 7.1:** For upper and lower dentures, scrape off any excess mixture overlapping onto the teeth of the denture.

Picture 7.2: For upper dentures only: cut excess mixture off the end of the denture downwards using a sharp knife.

Wait 10 minutes, before placing denture in the mouth. After the 10 minutes has elapsed, rinse the denture under running water for approximately 1 minute.

Your application is now complete. Re-apply every 30 days. **Do NOT** leave the product on your denture for longer than 30 days otherwise it becomes hard and difficult to remove. To remove the product from your denture see Point 1 above. You may eat, drink and clean your denture as per normal with the product. For after application care, see points listed above under the heading “**AFTER APPLICATION CARE**”.

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